

1. Download the App.

Search "BabyTempDuo" on Google Play or Apple App store and then install this app to yourphone.



2. User Login

Create an user login or recover your forgotten password on the intro page. You can follow the below sequence to register an account, log in or change the password.

The first screen is the login page. It features the "BabyTemp" logo at the top. Below it are two input fields: "Nickname" and "Password". There are two buttons: a green "Sign in" button and a blue "Join us free" button with an envelope icon. At the bottom, there is a link that says "Forget your password?".





The second screen is the registration page. It has a back arrow at the top left and the "BabyTemp" logo. It contains five input fields: "Nickname", "E-mail", "Birthday", "Password", and "Re-type Password". At the bottom, there is a blue "join us free" button.



The third screen is the "Change password" page. It has a back arrow at the top left and the "BabyTemp" logo. It features an input field for an email address with the example "sfalsdf@info.com" and a "SEND" button. Below that is an input field for an "Email verification code". At the bottom, there is a blue "Next" button.


3. App home page.

This page displays current measurements and historical data. Click each historical data, you can track its measurement report. Or you can click on the curve icon at the top right of the screen to see a graph of temperature changes.

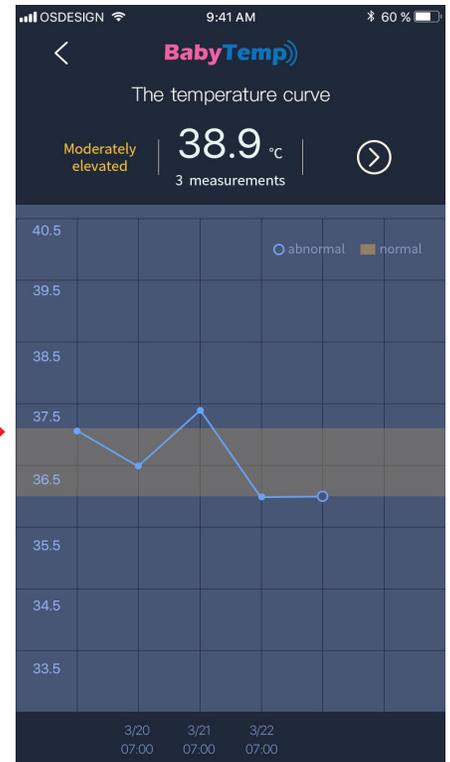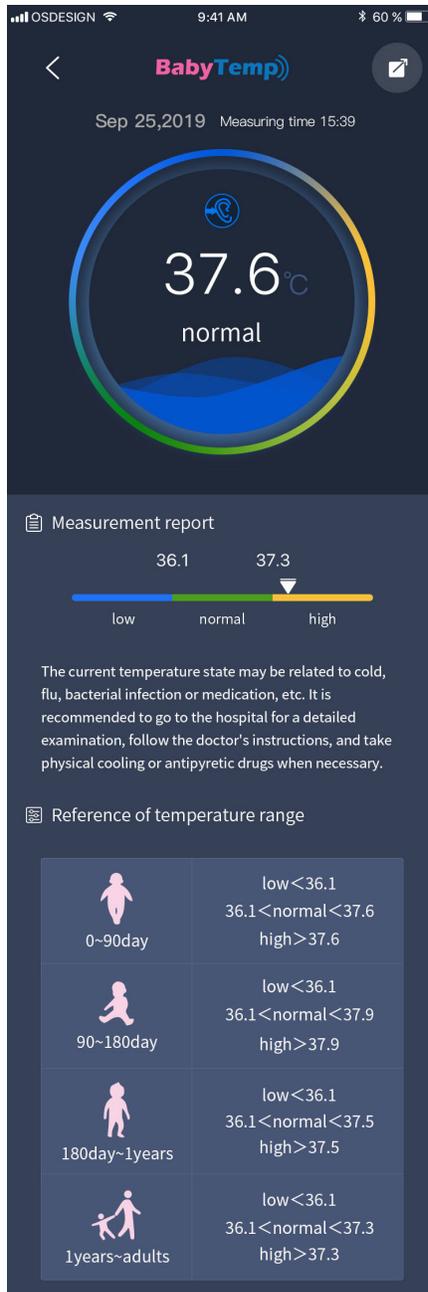




4. Manual Input

You can manually add temperature readings you want to save. To share this data, simply take a screenshot and send by text or email.

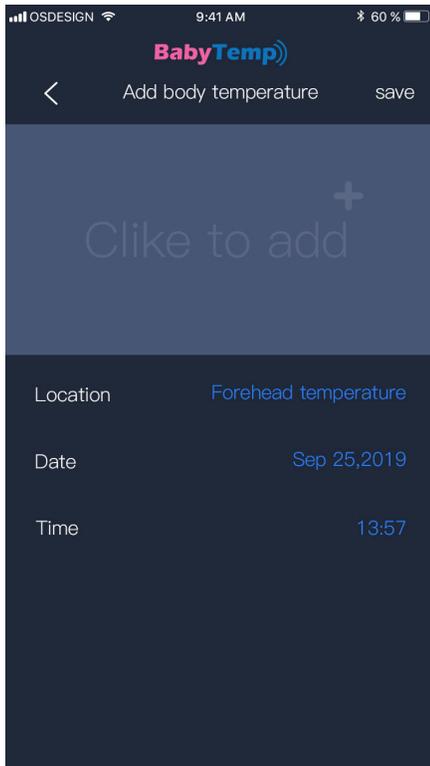

5. Multiple User History

Click the user name on the top left corner, add as many user profiles as needed simply by entering a name/nickname and birthday

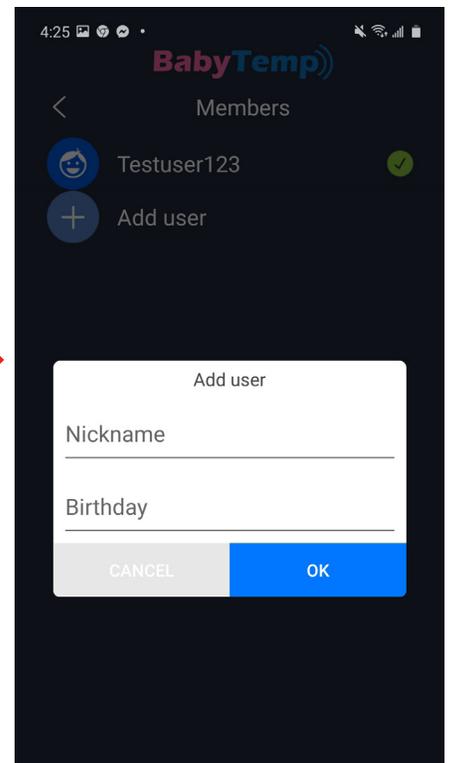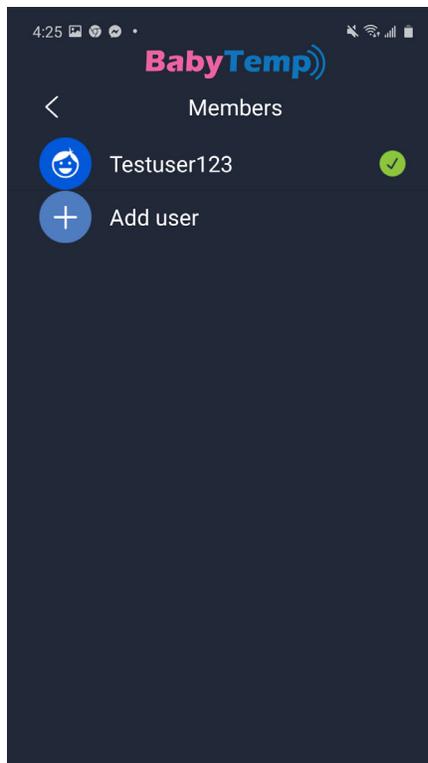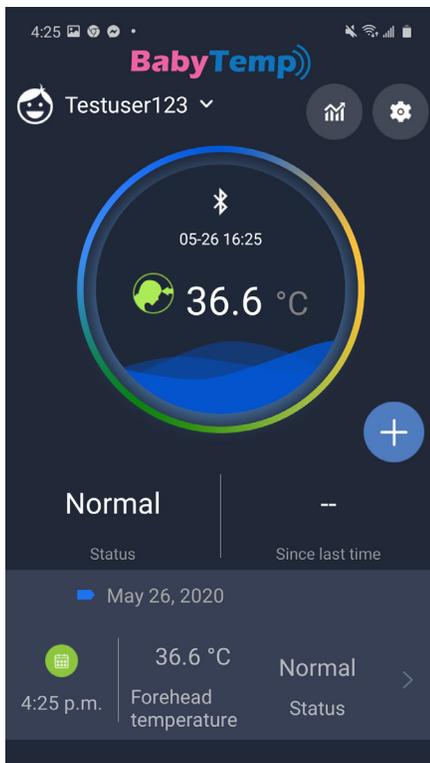

6. Settings

Click the gear icon at the top right corner of the screen to configure App settings.

"Switch Unit" allows users to choose between Fahrenheit and Celsius depending on preferences

"About" displays the App firmware version

"Account Info" allow users to change user name and password, but requires email verification for security purposes

"Help" brings the user to these set of instructions to help with App navigation

